




PRESTAN®

PRESTAN Professional Female Manikin

Replacing Adult Skin with Female Skin

Step 1: To remove the Adult Skin, lay manikin on its back.

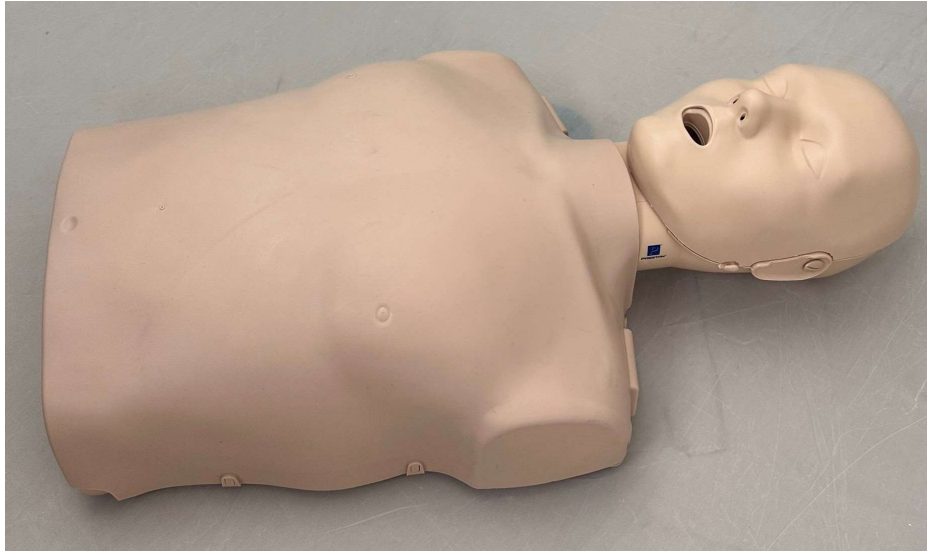

Step 2: Gently pull the Adult Skin off the torso studs (four at the shoulders, two on each side, and two at abdomen).





PRESTAN®

PRESTAN Professional Female Manikin

Replacing Adult Skin with Female Skin

Step 3: Remove the Adult Skin from the manikin.



Step 4: Place the Female Skin over the torso frame.






PRESTAN®

PRESTAN Professional Female Manikin

Replacing Adult Skin with Female Skin

Step 5: Gently attach the Female Skin back onto the torso studs (four at the shoulders, two on each side, and two at the abdomen).

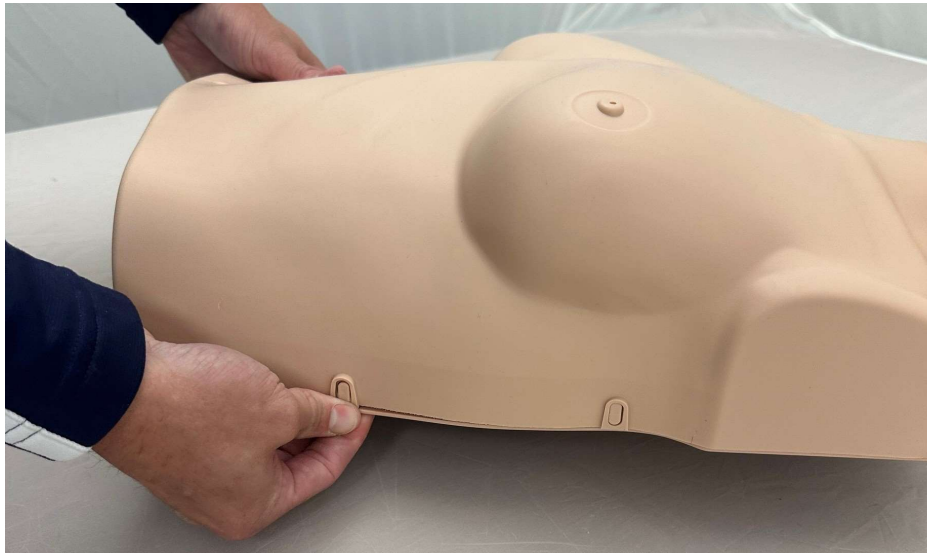

Step 6: The Female Manikin should look like this when complete.

