



PRESTAN®

PRESTAN Professional Adult Series 2000 Manikin and PRESTAN CPR Feedback App Help Guide

Connecting a Series 2000 Manikin to the PRESTAN CPR Feedback App

- Open the PRESTAN CPR Feedback App.
- Press and hold down on manikin for one full compression until the manikin beeps and all four lights illuminate in the shoulder.
- A single red light will continue to flash indicating that the manikin is on, however, the manikin is not yet connected to the app.
- Select 'Start a Training Session' in the app. The app will begin 'Searching for Manikins' and available manikin(s) will be listed on the 'Select Manikins' page.
- Click on a manikin and select desired color & number. Select manikin and click 'continue'.
- Once the manikin has been connected to the app, lights will scroll in the shoulder of the manikin.

Find My Manikin Feature

- Select 'Start a Training Session' in the app.
- Available manikin(s) will be listed on the 'Select Manikins' page.
- Select a manikin by clicking on the three dots to right.
- Select a color and number for the connected manikin.
- Select 'Find Manikin' - the connected manikin will continue to beep every five seconds; or a beep will be heard when the 'Find Manikin' button is selected. Note: beeping will not stop until you go back to the 'Select Manikin' page.





PRESTAN®

PRESTAN Professional Adult Series 2000 Manikin and PRESTAN CPR Feedback App Help Guide

How to Turn Off the Series 2000 Manikin

- Close out of the PRESTAN CPR Feedback App.
- After 10 minutes, the manikin will go into sleep mode and turn off.
- Note: If the app remains open, the manikin will stay on.

Meaning of the Series 2000 Manikin CPR Rate Monitor LED Sequence

	Connected Mode	Disconnected Mode
Sleep Mode/No Power		
Smart Device Connected Mode <small>(Cycles in sequence left to right)</small>		n/a
1st Full Compression, Not Connected Mode <small>(Wake mode/LED Test Mode)</small>	n/a	
CPM* < 60		
CPM* 60 – 79		
CPM* 80 – 99		
CPM* 100 - 120		
CPM* > 120		
Compressions Stopped, Not Connected Mode		
Compressions Stopped, Not Connected Mode Low Battery Mode		
Compressions Stopped, Connected Mode		

*CPM = Compressions per minute
 = Flashing

