

PRESTAN Professional Adult/Child Manikins

Frequently Asked Questions



Can you use a Pocket Mask with the new PRESTAN Professional Manikin?

The pliable face skin on the PRESTAN Professional Manikin makes it ideal for use with a Pocket Mask.

Can you use a Pocket Mask with a BVM on the new PRESTAN Professional Manikin?

You can use either a Pocket Mask alone or a Pocket Mask with a BVM on the PRESTAN Professional Manikin.

The CPR Rate Monitor lights on my adult/child manikin are not working properly.

There are a few things to check when your monitor lights are malfunctioning:

1. Make sure you are compressing fully, you should hear the clicker when fully compressed.
2. Ensure two AA batteries are installed in the battery compartment behind the left shoulder of the manikin. Replace batteries and recheck.
3. The clicker, which drives the monitor lights, may be malfunctioning. Contact your distributor for a replacement clicker.

How do you replace the batteries?

The compartment for the batteries is in the CPR rate monitor, located on the upper left side of the back of the manikin. Once the batteries are replaced, the lid is easily re-inserted by lining up the two tabs on the top of the compartment and pushing the lid into place.

I have a 4-Pack of Adult/Child Manikins. What is the proper way to pack them into the carry bag?

Please refer to the PRESTAN Professional Manikin Repacking Help Guide in the PRESTAN Resource Center.