



Prestan Professional Manikins

Care and Cleaning Recommendations

Cleaning:

- The face and the body skin of the Prestan Professional Manikin is made out of the same material and can be cleaned in the same manner
- If the material becomes dirty, clean with a mixture of soap and water. If the area is extremely dirty, common household cleaners such as Formula 409®, Windex® or Vim may be used. The material may also be conveniently wiped down with a Clorox® or Lysol® disinfecting wipe. It is recommended that the use of these cleaners be done cautiously with minimum amount of abrasion to insure the life of the skin.
- Don't forget to wipe the upper chest foam & skin in the same way in order to sanitize the inside of the manikin.
- The face skin may also be sanitized with alcohol wipes or soaked in a solution of 1/4 cup bleach mixed with one gallon of water for 10 minutes. Between student uses, the instructor (or student) may clean the faces off with an alcohol wipe as an extra measure of safety. Replacement face skins may be ordered from your Prestan Authorized Distributor.
- When using more than one student per manikin, we suggest each student have their own face shield which can be placed over the attached face-shield/lung-bag when administering breaths. Face shields may be ordered from your Prestan Authorized Distributor.
- Lung bags should be replaced after each class as bacterial moisture (from giving breaths) is trapped inside. Replacement lung bags may be ordered from your Prestan Authorized Distributor.

Storage:

- Store your Prestan Professional Manikin in its original carrying case. Further literature is available on proper repacking of the manikin carry bags.
- Remove the heads from the torsos for extra protection.
- Do not stack the manikins as this may damage the noses.
- Do not store the manikins in a damp area or in a place with extreme temperatures.

