





PRESTAN AED Raizor Trainer

Setup

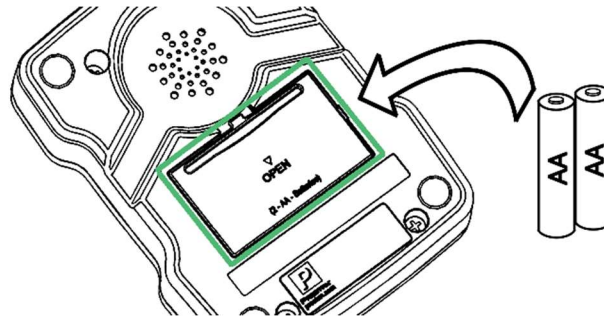

Insert 2 AA Batteries (Not Included):

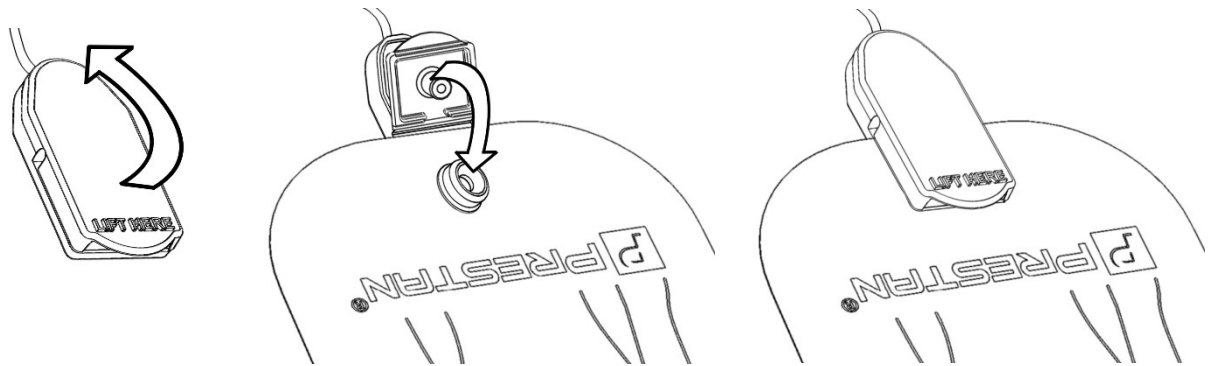

Attach Pads to Cable – **Do NOT insert cable to unit until instructed by trainer:**



Controls

	Press to start training
	Quick tap to pause during training Quick tap again to resume.
	Hold down to power off.
	Use only during training when instructed.

Scenario Details

- The unit is pre-programmed with a single scenario – Shock, Shock, No-Shock
 - Shock - Device begins with analyzing patient, shock requested.
 - Shock - 2 minutes of CPR, followed by analyzing patient, shock requested.
 - Next 2 minutes of CPR – No-Shock requested.
- A count is provided to help with CPR timing. The count will announce the 10th, 20th and 30th expected compression.

Best Practices

- Remove batteries for storage longer than 3 days.
- Replace pads on liner for storage. Pads can be reused multiple times, depending on cleanliness. Once pads no longer adhere, dispose of them properly and use a new set. New pad sets are available from your distributor (Part Number PP-RTPAD-05).
- Clean unit with dry or slightly damp cloth.
- Do not allow liquids on or near the device or the pads.