



**PRESTAN®**

## **PRESTAN Professional AED Trainer PLUS and PRESTAN AED UltraTrainer**

How to toggle between 15:2 and 30:2 compression-ventilation ratio

**Step 1:** Hold child button and power button down at the same time.  
The unit will then beep.

**Step 2:** Release both buttons at the same time.

**Step 3:** Push child button to select your preferred option (15:2 or 30:2).

**Step 4:** Once you have selected your preferred option, power unit off.

**Step 6:** Turn unit back on and you're ready to train.

