



PRESTAN[®] PRO⁺ ADULT HEAD ASSEMBLY




LUNG BAG INSTALLATION

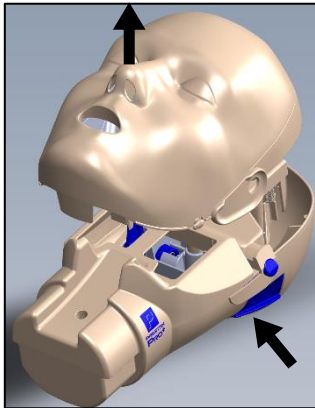

1.) Squeeze both blue buttons on back of head and lift off Face assembly.

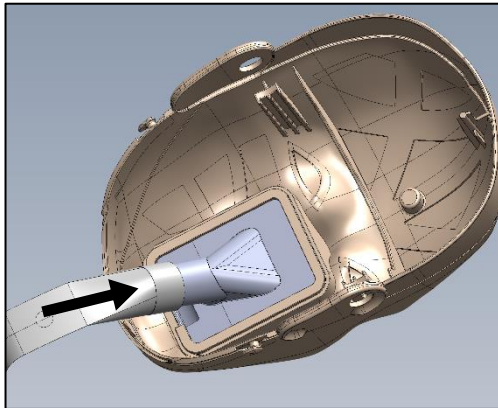


2.) On back of face, press fit Lung Bag ring onto Oral/Nasal Insert tube.

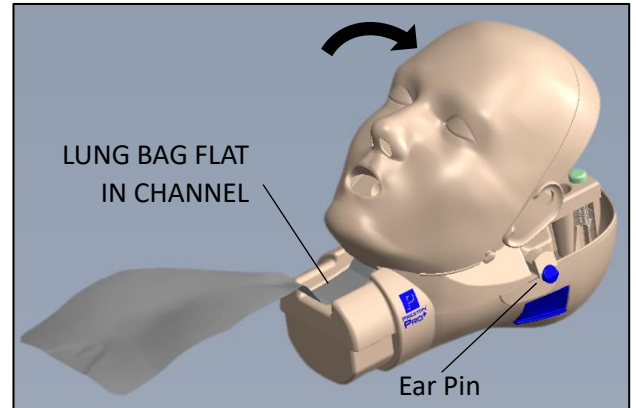


3.) Attach Face assembly onto Head Back by aligning first at chin and rocking back to engage ear pins. Confirm ear pins are fully engaged and Lung Bag is flat in neck channel.


REPLACING ORAL/NASAL INSERT

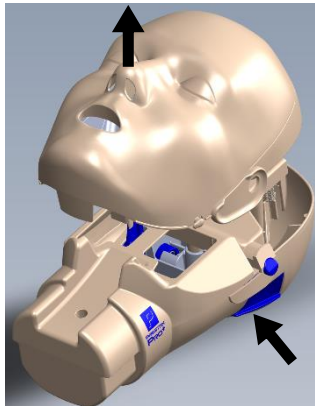


1.) Squeeze both blue buttons on back of head and lift off Face assembly.

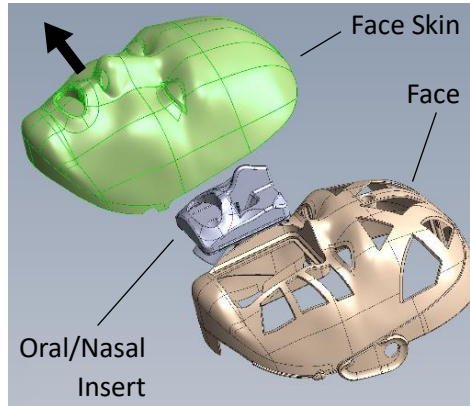

2.) Remove used Face Skin and Oral/Nasal Insert from Face.

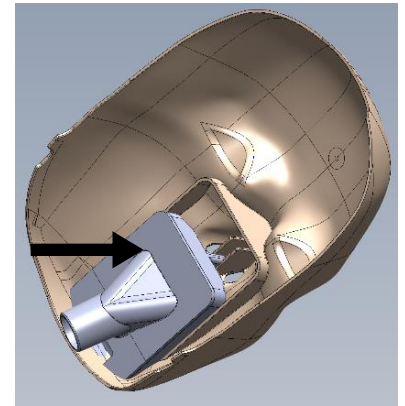

3.) Insert new Oral/Nasal Insert into pocket inside Face Skin

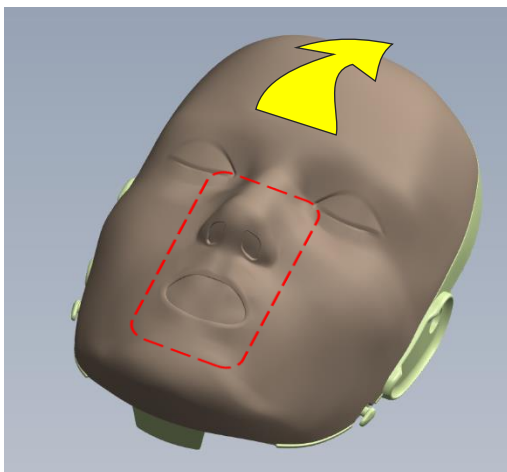

4.) Attach face skin assembly from step 3 to face ensuring that the Oral/Nasal insert is aligned with the opening in the face. Seat the assembly by pressing down along the red dotted line shown here. Confirm outer edges of Face Skin are secure by running your thumbs from the brow of the face to the crown of the head using gentle pressure to slightly stretch the skin over the face.

